

Instructions


With her bodywear, Javita aims to promote safe PD catheter use and ensure maximum wearing comfort for our users. Instructions for use are important and that is why we provide them in different languages for prescribers, caregivers, users and interested parties.



1.



2.

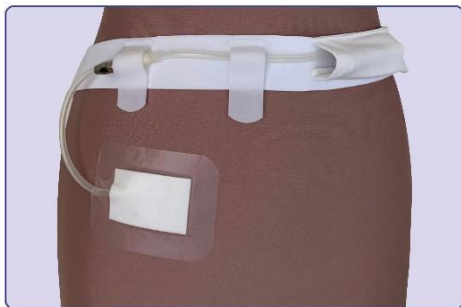

3.

1. Attach the belt around the waist with the velcro strap
2. Store the end of the Miniset in a case
3. Close the velcro so that the Miniset is fixed

Technical information

The belt consists of an elastic strap made of polyamide and elastane. The velcro straps are made of polyester. The pouch, in which the Miniset can be stored, is made of organic cotton.

Washing instruction

The PD-Belt can be washed at a temperature up to 40°C and should not be dried in the dryer. The velcro strap should be closed properly during washing.



Do you have any questions?

Check the website at our Frequently Asked Questions (FAQ) section. Many questions have already been answered. Any other questions?

Send an email to info@javita.nl.

Feedback

If you would like to share your experience and/or feedback with us, we would love to hear from you. You can send us an email to info@javita.nl. Your response is very valuable to us and is greatly appreciated, so that we can respond even better to the wishes and needs of our users.

Size	Waist size	
	cm	inch
XXS	55 – 60	22 – 24
XS	60 – 65	24 – 26
S	65 – 80	26 – 32
M	80 – 95	32 – 37
L	95 – 110	37 – 43
XL	110 – 130	43 – 51
XXL	130 – 150	51 – 59